



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER IMMOBILIZERS

ULTRA SHOULDER ABDUCTION PILLOW W/SLING

- > Features extended pillow design with malleable stay for proper positioning and V-lock strapping system to prevent rolling and shifting
- > Indicated for treatment after Bankart repairs, rotator cuff repairs, shoulder arthroscopy, separations and dislocations
- > Constructed of firm density foam covered in tricot material with tricot sling and exercise ball to stimulate circulation



PRODUCT NUMBER

S	M	L	XL
23-1901	23-1902	23-1903	23-1904

SIZING GUIDE

	S	M	L	XL
Hip	24"-30"	30"-36"	36"-42"	42"-48"
Forearm	9"-12"	12"-15"	15"-18"	18"-20"

Measurement taken from hip circumference and length of forearm from wrist to elbow

APPLICATION INSTRUCTIONS

1. Separate sling from pillow and remove 1 1/2" x 16" secure strap from sling.
2. Detach shoulder strap and open front panel.
3. Place thumb through loop inside sling for additional comfort.
4. Pass shoulder strap diagonally up and over the unaffected shoulder and fasten through both buckles so the end of the strap is at mid chest. The shoulder strap length can also be adjusted at the back of the sling.
5. Place pillow at waistline with large end facing forward. Change angle of pillow by bending malleable stay if desired. Attach plastic buckle and adjust length of the strap at the back end of pillow. Attach sling to the pillow, lining up hook and loop contact closure for a secure fit.

Note: The pillow is designed to keep the arm in approximately 15 degree of abduction. The position of the pillow at the waist should be positioned for desired internal or external rotation by sliding the pillow forward or back along the waistline.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.